ardo amaryll manual breastpump

Instructions for use | Gebrauchsanweisung | Mode d'emploi |
Istruzioni per l'uso | Instrucciones de uso | Indicações de
utilização | Gebruiksaanwijzing | Bruksanvisning | Käyttöohje |
Bruksanvisning | Brugsanvisning | Инструкция по эксплуатации |
Instrukcja obsługi | Οδηγίες χρήσης | Kullanım Kılavuzu |
دليل الاستعمال | 사용 설명서 | 使用说明

	Languages		Page
en	Instructions for use	Amaryll manual breastpump	4–7
de	Gebrauchsanweisung	Amaryll Handmilchpumpe	8–11
fr	Mode d'emploi	Tire-lait manuel Amaryll	12–15
it	Istruzioni per l'uso	Tiralatte manuale Amaryll	16–19
es	Instrucciones de uso	Sacaleches manual Amaryll	20–23
pt	Indicações de utilização	Bomba de extração de leite manual Amaryll	24–27
nl	Gebruiksaanwijzing	Amaryll handkolf	28–31
sv	Bruksanvisning	Amaryll handbröstpump	32–35
fi	Käyttöohje	Käsikäyttöinen Amaryll-rintapumppu	36–39
no	Bruksanvisning	Amaryll håndpumpe	40–43
da	Brugsanvisning	Amaryll håndbrystpumpe	44–47
ru	Инструкция по эксплуатации	Ручной молокоотсос Amaryll	48–51
pl	Instrukcja obsługi	Laktator ręczny Amaryll	52–55
el	Οδηγίες χρήσης	Χειροκίνητο θήλαστρο Amaryll	56–59
tr	Kullanım Kılavuzu	Amaryll Manuel Süt Pompası	60–63
ar	دليل الاستعمال	مضخة اللين اليدوية Amaryll	64–67
ko	사용 설명서	애머리III 수동유축기	68–71
zh	使用说明	爱瑞手动吸乳器	72–75

1. Package contents

Amaryll manual breastpump:

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Amaryll Kombikit: from a to d

2. Indications

The Amaryll manual breastpump is used for expressing, collecting and storing breast milk.


3. Instructions for use

3.1 Assembly

1. Assemble according to the illustration
2. Place the membrane pot securely over the groove of the membrane con rod. Check that the smooth side of the membrane pot faces outwards.
3. Put on the casing cover in the desired position:
 - 3.1 Standard position
 - 3.2 Lever position left side
 - 3.3 Lever position right side


The mark on the casing cover must lie on top of the mark on the breast shell. Then rotate the casing cover clockwise.

4. The lever is mounted on the membrane con rod and by means of light pressure downwards clicks into the casing cover.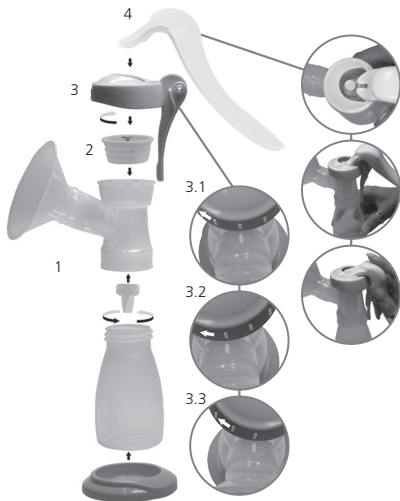

3.2 Expressing milk



- Wash your hands, sit down comfortably and relax.
- Place the casing cover in the desired position and tighten it.
- Centre the nipple so that when pumping it moves freely in the breast shell/breast shell insert and pump by activating the lever.
- Adjusting the suction intensity (vacuum): The further you press the lever, the greater the suction intensity. **1**
- Adjusting the speed interval: The faster you move the lever up and down, the higher the pumping frequency. **2**
- Our recommendation: Begin at a high speed interval and at low suction intensity in order to stimulate the milk ejection reflex. Once the milk is flowing, reduce the speed interval and increase the suction intensity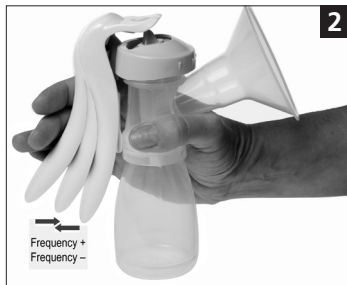

3.3 Choice of breast shell and Optiflow massage insert

If your nipple rubs uncomfortably on the inner side of the funnel, use a larger breast shell/breast shell insert. If the areola is also sucked into the funnel, use a smaller breast shell. The Optiflow massage insert can only be used with the 31 mm breast shell.

28 mm breast shell insert Optiflow

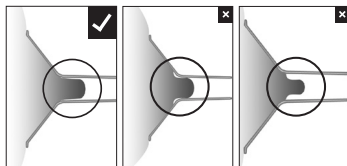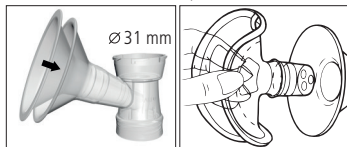4. Storage of breast milk

Recommended storage time at room temperature: 3–4 hours at 16–29°C (60–85°F); in the refrigerator: 3 days at ≤4°C (39°F); in freezing compartment with door: 6 months to 1 year at <-17°C (0°F)

5. Cleaning

Manual cleaning

Before using for the first time and after each time it is used:

- Dismantle the breastpump into its individual parts.
- Rinse for at least 1 minute using cold tap water.
- Clean the components of the pump with commercial detergent and warm water (not over +40°C / 104 °F) and rinse with cold tap water. Make sure that all detergent residues are removed.
- A soft brush can also be used for heavy soiling.

ATTENTION! Do not clean the lip valve with the brush.

- Allow all parts of a clean surface to air dry.

Manual disinfection

1x daily after manual cleaning

- Make sure there is enough water in the pot in order to prevent damage/deformation.
- Boil the parts for 5 minutes.
- Allow the parts to cool down and dry on a clean surface.
- The lever, casing cover and bottle stand can, but need not, be boiled.

Check the individual parts for cracks and changes to the surface according to the cleaning and disinfection process. If you discover any changes, dispose of the part affected and replace with a new one.

Function test

Once assembled, the Amaryll is placed on a hairless area of the skin (stomach area). When the breast shell is fully in place, a vacuum should be perceptible when the lever is pressed. Once this is done, the Amaryll can be used again.

6. Possible problems and troubleshooting

Weak or no vacuum:

- Ensure that the Amaryll manual breastpump has been correctly assembled and is not damaged.
- Check whether the lip valve is cracked or stuck together.
- Make sure that the casing cover seals well.

Pain when pumping:

- Ensure that the nipple is in the middle of the breast shell.
- Check that the breast shell size fits your nipples.
- If problems continue seek professional help.

7. Contraindications

None known.

Safety regulations: Only use after cleaning or sterilisation. After cleaning check for any damage.

8. General information

	Materials: Polypropylene (PP) / silicone		Humidity limitation: 20–95%
	Temperature limit: Operation -10°C (14°F) to +40°C (104°F) / Transport -20°C (-4°F) to +50°C (122°F)		Atmospheric pressure limitation: 900–1060 hPa
	Disposal: Can be safely disposed of in compliance with local regulations.		Keep away from sunlight
	Fragile, handle with care		Keep dry

Preferably store Optiflow in a dark place (silicone).

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Ardo medical AG