 ardo

0123

Languages:

Page:

de	Gebrauchsanweisung Glass Bottle	2
en	Instructions for use Glass Bottle	3
fr	Mode d'emploi Glass Bottle	4
it	Istruzioni per l'uso Glass Bottle	5
nl	Gebruiksaanwijzing Glass Bottle	6
es	Indicaciones Glass Bottle	7
pt	Indicações Glass Bottle	8
tr	Kullanım kılavuzu Glass Bottle	9
da	Brugsanvisning Glass Bottle	10
sv	Bruksanvisning Glass Bottle	11
no	Bruksanvisning Glass Bottle	12
fi	Käyttöohjeet Glass Bottle	13
pl	Instrukcja Glass Bottle	14
el	Οδηγίες χρήσης Glass Bottle	15

Glass Bottle

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Glass Bottle

For use in clinics and hospitals

for the collection and storage of breast milk.

How to use these bottles

Expressing breast milk using a breast pump.

Connect the bottle to an original Ardo Pumpset (see separate Instruction leaflet).

Storage of breast milk

Seal the bottle with the lid provided.

Breast milk can be stored as follows:

At room temperature: 6 hours at max. 25 °C; in the fridge: 5 days at max. 4 °C; in the fridge-freezer compartment, with door: 3 - 6 months; in the deep freeze: 6 - 12 months at -20 °C

Cleaning

Manual cleaning

Before using for the first time and after each time it is used:

- Rinse for at least 1 minute using cold tap water.
- Clean the components of the bottle with commercial detergent and warm water (not over +40°C / 104 °F) and rinse with cold tap water. Make sure that all detergent residues are removed.
- A soft brush can also be used for heavy soiling.
- Allow all parts of a clean surface to air dry.

Manual disinfection

1x daily after manual cleaning

- Make sure there is enough water in the pot in order to prevent damage/deformation.
- Boil the parts for 5 minutes.
- Allow the parts to cool down and dry on a clean surface.

Check the individual parts for cracks and changes to the surface according to the cleaning and disinfection process. If you discover any changes, dispose of the part affected and replace with a new one.

Material

Glass; Polypropylene

Disposal

All parts must be disposed of in accordance with local regulations.

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Ardo medical AG