
## Instructions for use Tulips Nipple shields

en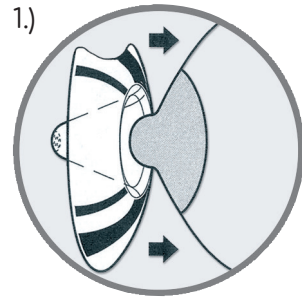

**Intended use:**  
The Tulips nipple shields are used during breastfeeding to create a protective, perfectly shaped contact surface between the nipple and the baby's mouth.

**Indication:**  
The Tulips nipple shields are ideal for temporary use with sucking problems and sore, sensitive nipples during breastfeeding.

**Contraindication:**  
There are no known contraindications.

### Use:

- Always wash hands thoroughly with soap and water before use.
- Moisten the inside of the nipple shield with a little breast milk or water, so that it adheres more easily to the breast. Pull back the edge of the nipple shield and centre the shield over the nipple. The baby's nose should be at the tulip-shaped cut-out, so that it is touching the mother's skin (image 1).
- Release the edges of the nipple shield onto the breast (image 2).

### Warnings:

The use of nipple shields

- may delay the let-down reflex
- may impair lactation over the long term as the nipples are stimulated less during breastfeeding
- may make the baby reluctant to latch on without the nipple shield. Breastfeed regularly without using any aids.

Ardo recommends temporary use of the nipple shields and support by a professional.  
Store the product out of reach of children.

### Manual cleaning

Before using for the first time and after each time it is used:

- Rinse for at least 1 minute using cold tap water.
- Clean the parts with commercial detergent and warm water (not over +40°C/104°F) and rinse with cold tap water. Make sure that all detergent residues are removed.
- Allow all parts of a clean surface to air dry.

### Manual disinfection

1x daily after manual cleaning

- Make sure there is enough water in the pot in order to prevent damage/deformation.
- Boil the parts for 5 minutes.
- Allow the parts to cool down and dry on a clean surface.

Check all parts for cracks and changes each time you clean and disinfect them. Dispose of and replace damaged parts immediately.

### Disposal:

Harmless, in accordance with local regulations.

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