easy cup

Instructions for use | Gebrauchsanweisung | Mode d'emploi |
Istruzioni per l'uso | Instrucciones de uso | Indicações de utilização |
Gebruiksaanwijzing | Bruksanvisning | Käyttöohje | Bruksanvisning |
Brugsanvisning | Instrukcja obsługi | Οδηγίες χρήσης | Kullanım
Kılavuzu

ard
SWITZERLAND

Easy Cup

The alternative feeding method

The Easy Cup can be used by specialists and parents as an alternative feeding method to breastfeeding. It is used for example for babies who have problems with sucking and latching-on in the first few days, or for children with a cleft lip and palate and for weak, sick or prematurely-born children. As far as possible breast milk is always administered. The baby will drink best if it is in a sprightly and inquisitive mood.

Caution: The risk of aspiration exists if the liquid is poured directly into the mouth of the baby.

Application

1. Sit down in an upright position, comfortably and relaxed.
2. Fill the Easy Cup if possible, at least up to half-full. Put a cloth under the baby's chin. Restless babies can be wrapped up in a towel.
3. It is imperative to hold the baby in an upright sitting position on your lap, so that you and the baby feel comfortable.
4. Place the lower rim of the Easy Cup gently against the lower lip of the baby. Do not exert any pressure on the lower lip.
5. Tip the beaker so that the milk touches the baby's lower lip. NEVER pour the milk directly into the mouth.
6. If the baby takes a break in drinking, leave the Easy Cup touching the lower lip.
7. If feeding is finished, allow the baby to belch.

Cleaning

- Cleaning in the clinic: Sterilisation at 121 °C = 20 Min. / 134 °C = 5 Min.
- At home: Prior to initial use and once daily boil out for 10 minutes or put in the dishwasher or steam steriliser. Dry off with a clean cloth and cover. None of the parts are suitable for the microwave.

Material: Beaker and lid consist of BPA-free polypropylene (PP)

General information

	Temperature: Operation +5 °C (41 °F) to +40 °C (104 °F) / Transport –20 °C (–4 °F) to +50 °C (122 °F)		Relative humidity: 20–95%
	Disposal: Can be safely disposed of in compliance with local regulations.		Keep packaging away from heat
	Packaging: Caution, fragile		Keep packaging away from moisture

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Ardo medical AG